XCellarisPRO MC

Medizinprodukt gem. Richtlinie 93/42/EWG – MADE IN GERMANY

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XCellarisPro MC

Medical device acc. to Directive MDD 93/42/EEC – MADE IN GERMANY

Please read the instructions carefully.

They contain important information on how to handle the microneedling device XCellarisPro MC.

Intended purpose: Minimally invasive perforation of the dermis (microneedling)

Effect: Microneedling causes minimally invasive micro-injuries of epidermis and dermis, without causing open wounds in the classical sense. The puncturing channels close after a short time. The micro-injuries trigger the body's wound healing cascade, which, inter alia, induces an increased collagen production in the treated skin. For this reason, the method is also called "percutaneous collagen induction" (PCI) or "Collagen Induction Therapy" (CIT).

Medical indications:

- To treat and repair atrophic and hypotrophic scars including acne scars, mature hypertrophic scars resulting from injury, trauma or disease, burn scars, and stretch marks (*striae distensae*).

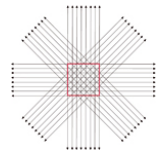
In general, the XCellarisPro MC is applicable to almost all skin areas of the patients and in all skin types. Exceptions are described under "Contraindications".

Precautions / Notes:

- The device is only to be used by trained professionals
- The treatment must be carried out under low-germ conditions and in hygienically suitable treatment rooms
- Sterile disposable gloves must be worn during the treatment
- Medical Device for **SINGLE USE only**. In case of reuse there would be a risk of skin infections!
- Reprocessing of the device is prohibited since this could lead to damage or quality deterioration
- Do not use if the sterile packaging is damaged
- Store at room temperature and away from heat sources
- After the expiration date, the instrument may not be used

Procedure:

- Clean the skin area to be treated, anaesthetise with a topical anaesthetic cream, then disinfect the skin prior to treatment
- Roll the same area about 6 times in each direction (as illustrated) until the treated skin shows a uniform mild redness and pinpoint bleedings appear.
- Apply only mild pressure and do not apply lateral pressure on the device
- Don't change rolling angle while the needles are in the skin
- In skin areas with thin dermis directly over bone (eg. periorbital) only very low pressure may be used

Post-treatment:

- Clean the treated skin area with sterile swabs and sterile saline solution

Treatment cycles:

- A sustained treatment success can be expected only after repeated treatments.
- Usually 3-5 treatments are carried out, but depending on scar depth and structure also more treatments may be necessary.
- The interval between two treatments should be at least 4-6 weeks to make sure that the wound healing cascade is completed.

Contraindications:

- Treatment of mucous membranes and the eye - the skin around the eyes should be treated with extreme caution
- Patients with or a history of keloids
- Treatment of skin areas with skin cancer, active herpes simplex, active skin infections, raised moles, warts, solar keratoses

Safety notes

- Though no clear contraindication for microneedling, special attention is advised in patients with increased bleeding tendency or wound healing problems, e.g. due to uncontrolled diabetes mellitus or anti-coagulant therapy, as well as patients receiving chemotherapy, radiotherapy or high doses of corticosteroids
- Pregnant / nursing women: since there are no clinical results on pregnant / nursing women, we recommend to postpone the treatment
- Children / adolescents: the clinical study results were obtained mainly in adult volunteers. Thus, the treatment is generally recommended to be used in patients "over 18 years". However, there is no evidence that the treatment in adolescents and children constitutes an increased risk to the safety of the patient. Especially in treatment of burn scars in children, good results have been produced with the identical medical device Dermaroller MC.
- When combining the microneedling treatment with other therapies, the other manufacturer's instructions for use have to be followed

Accessory symptoms of the treatment:

Transient redness and swelling of the treated skin, pain, microbleedings for minutes

Potential adverse effects:

Skin infections due to inadequate disinfection, inflammatory response (caused by pyrogens), transient pigmentation disorders, scarring when a tendency to keloid formation is present, granuloma after the use of active substances during the XCellarisPro treatment.

A good skin hygiene before and after the procedure is advised to further minimise the risk of infection.

Prolonged sun or other UV exposure is not recommended after the procedure for at least 8 days to minimize the risk of hyperpigmentation.

Patients should use a sun screen that provides a high level of UVA and UVB protection.

The stainless steel of the XCellarisPro microneedles contains ca. 8% nickel. Due to the short treatment duration, it is very unlikely that a patient develops an allergic reaction.

Disposal

Used XCellarisPro devices pose a risk of injury and a potential health hazard due to contact with blood. They are classified as „sharps“ (European Waste Catalogue 18 01 01) and must be collected in separate, closed containers and then disposed of safely.

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